
Care range

EN | User manual
FR | Mode d'emploi
ES | Instrucciones de uso
IT | Istruzioni per l'uso
DE | Bedienungsanleitung

NL | Gebruiksaanwijzingen
PT | Manual de instruções
PL | Instrukcja obsługi
HU | Használati útmutató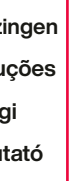

EN Ear spray Cerumenolytic

The spray is indicated to facilitate the elimination of earwax and to avoid its accumulation in the ear canal.

Odinell is a wash solution enriched with non-aggressive surfactants which facilitates the removal of earwax residue. Used regularly Odinell Spray prevents the formation of earwax plugs.

Instructions for use:



- 1 - Place the nozzle on the bottle.
- 2 - Position the nozzle at the entrance to the ear canal. Press the pump.
- 3 - Massage the base of the ear, then tilt the head to allow the liquid and any earwax residue to escape.
- 4 - Reapply if necessary and wipe with a clean, soft cloth.
- 5 - Clean the nozzle with warm water after each use

Directions for use: 1-2 sprays, 2 times per week.

In case of ear pain, consult your doctor. Not suitable for cleaning hearing aids. Keep out of reach of children. Do not place the nozzle too far inside the ear canal. This can cause irritation.

Contraindication: In case of otitis, tympanic membrane perforation, presence of tympanostomy tubes (grommets) or foreign bodies in the ear canal. On children under 36 months of age. After ear surgery.

If you experience problems, contact a hearing aid specialist. All serious incidents related to the device should be the subject of a notification to the manufacturer and the competent authorities in the Member State in which the user is located.

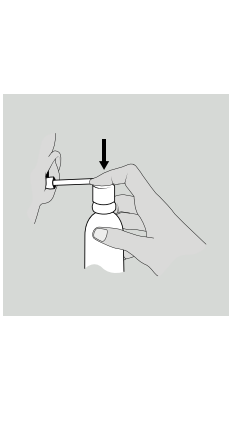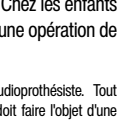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